



# Decompression Table

## Parts

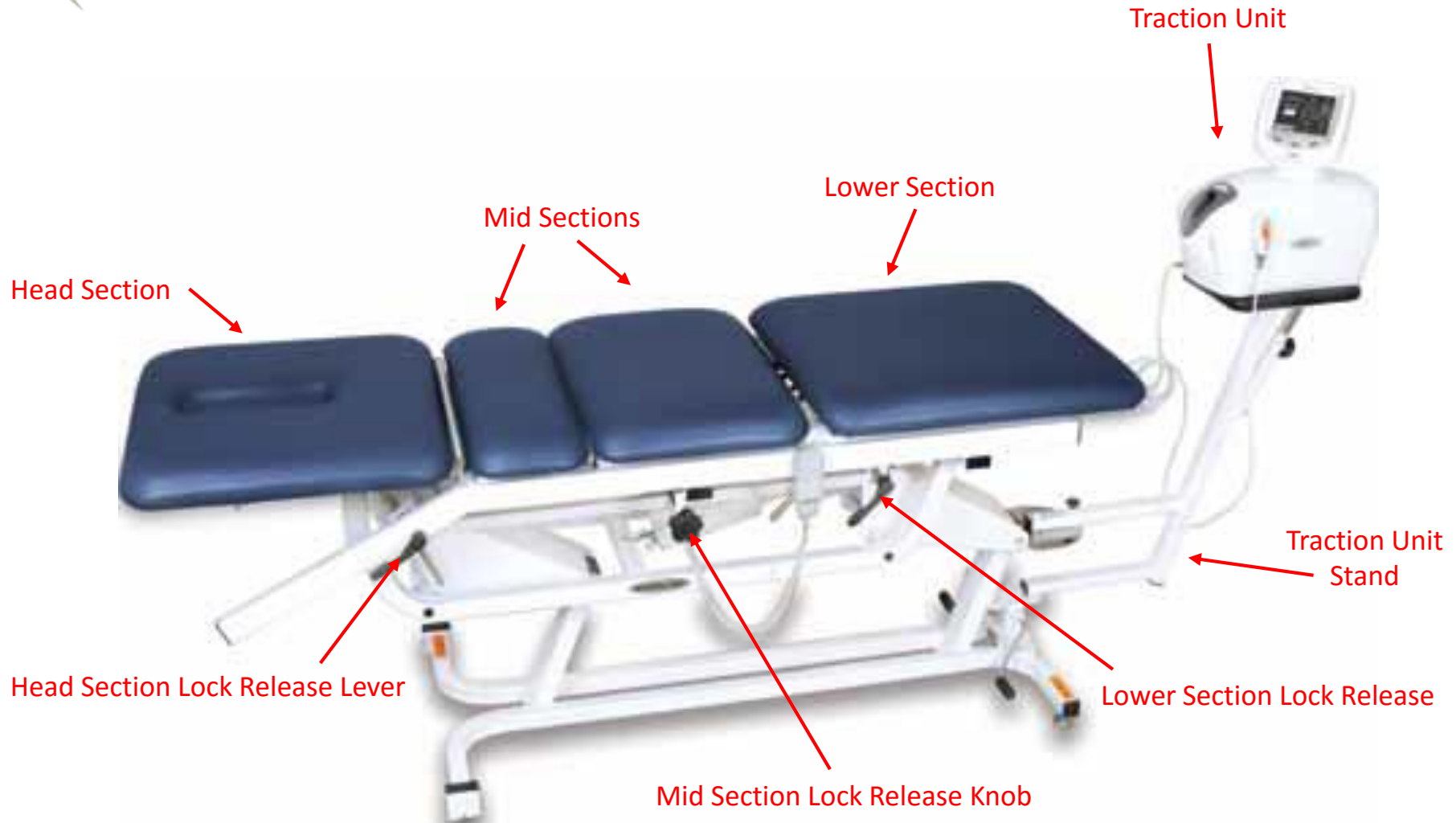

# Decompression Table Accessories

## Deluxe Universal Belt System

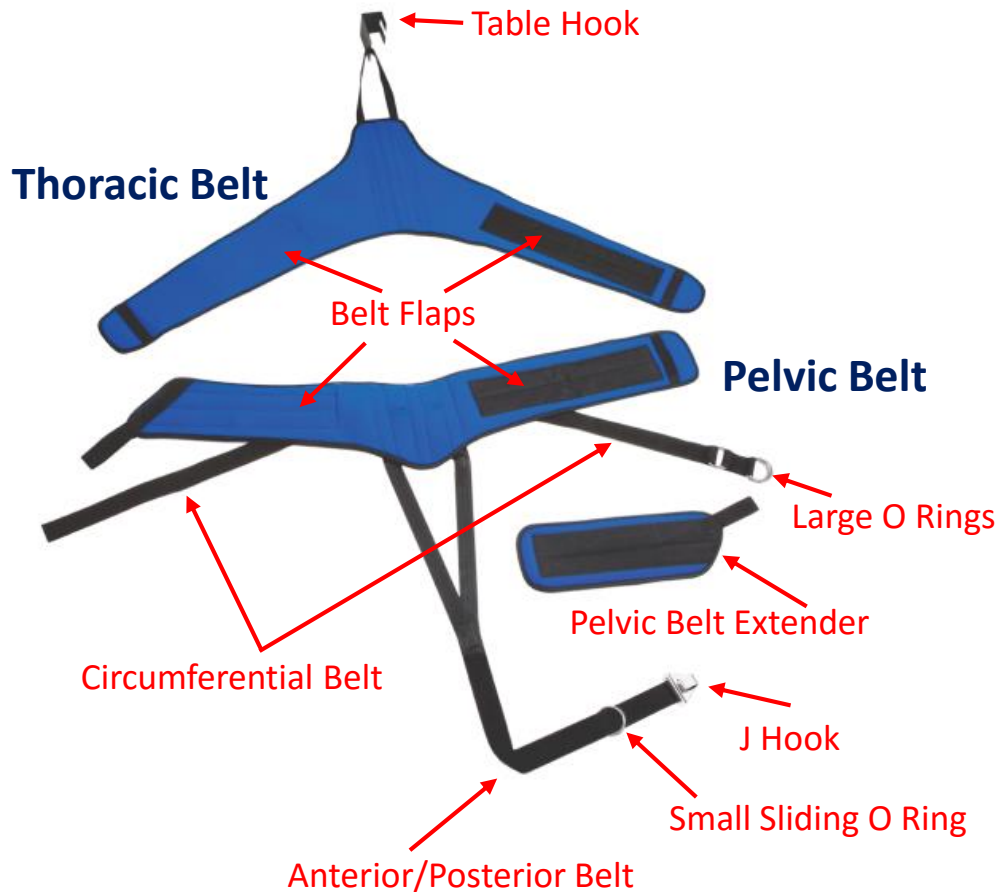

## Saunders Cervical Traction Device



# Decompression Table Usage Instructions

## WARNING:

- Before making adjustments to the table sections, make certain the patient is removed.
- Never place your hands and feet near the working mechanism of the table when making any and all adjustments to heights or table sections.

## Adjusting Table Sections

### Elevating Head Section

Raise the Head Section by hand to the desired elevation.

### Lowering Head Section

Lift the Head Section Lock Release Lever and lower the Head Section to the desired elevation. Support the Head Section when lowering.

### Locking Mid Section

Turn the Lock Release Knob toward the Lower Section of the table to lock the Mid Section in place.

### Unlocking Mid Section

Turn the Lock Release Knob toward the Head Section of the table to unlock the Mid Section.

### Elevating Lower Section

Raise the Lower Section by hand to the desired elevation.

### Lowering Lower Section

Turn the Lower Section lever and move the Lower Section down to the desired elevation.



# Decompression Table Usage Instructions

**WARNING:**

- Before making adjustments to the table sections, make certain the patient is removed.
- Never place your hands and feet near the working mechanism of the table when making any and all adjustments to heights or table sections.

## Table Height Adjustment

### Raising and Lowering Table with Optional Foot Pedal

Step on the corresponding side of the foot pedal to raise or lower the table to the desired height.



# Using the Belt System

## Attaching the Pelvic Belt

1. Place the belts on the table so that the blue side faces up, the Pelvic Belt overlaps the Thoracic Belt, and the black Table Hook is firmly seated to the head of the table.



2. Position the patient so the patient's waist is in the middle of the Pelvic Belt.



3. While maintaining tension on the first flap, wrap the second flap over the first flap, staying level with the patient's waistline. **NOTE:** The flaps should be exerting firm pressure around the patient's waist.



4. Loop the end of the Circumferential Belt through both Large O Rings and thread back into one of the Large O Rings and tighten.



5. To attach the Pelvic Belt to a traction unit, connect the J Hook to the Large O Ring.



6. Then, connect the rope from the traction unit to the Small Sliding O Ring.






# Saunders Traction Device

## Assembly Instructions

1. Attach the traction clevis bracket into the clevis attachment slots of the traction unit.

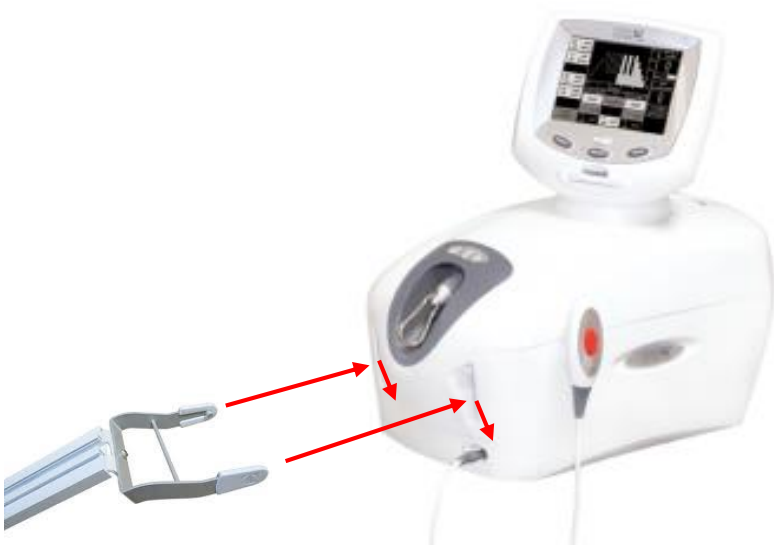

2. Attach the traction rope hook to the large hole at the top of the head pad.



# Saunders Traction Device

## Setup and Use Instructions

1. Adjust the angle of the device by raising or lowering the height of the table traction pedestal.
2. Separate neck wedges sufficiently by turning either knob on each side of the device.
3. Ask the patient to remove earrings (if any).

4. Position the patient in the device with the neck wedge located at the mid cervical region – about 1.5 inches below the occipital bone.
5. Tighten the neck wedge very firmly against the neck.
6. Manually rotate the neck wedges to provide optimum fit.

7. Secure the head strap.
8. Make sure the traction rope is hooked to the top of the head pad. Give the patient the treatment interrupt switch and instruct how to activate.

9. Program the traction unit and initiate treatment. During traction, if the patient's ears are being pushed up by the neck wedge, stop traction and return to step 4.

